



# BREEZESUITE™ 8.4 Service Pack 3 Release & Installation Notice

This software is a Service Pack for BreezeSuite™ cardiorespiratory diagnostic software version 8.4 and is intended to be installed on any computer that has BreezeSuite 8.4 already installed. BreezeSuite 8.4 Service Pack 3 can be installed on workstations, but not on BreezeSuite Servers.

## **Minimum System Requirements**

- BreezeSuite cardiorespiratory diagnostic software version 8.4

**Warning:** Installing Service Pack 3 over BreezeSuite software version 8.4 will over-write the translation files.

**NOTE:** For multiuser installations, Service Pack 3 is only applied to workstations. Service Pack 3 is not applied to the BreezeSuite Multiuser™ Server installation.

## **INSTALLATION**

1. Close all applications.
2. When downloaded from the MGC Diagnostics website, extract/unzip the BreezeSuite84SP3.zip file.
3. Double click on the BreezeSuite84SP3.exe file from the extracted files downloaded from the MGC Diagnostics website.
4. After the software installation is run, it will begin to install the appropriate files to your hard drive.

## **LANGUAGE TRANSLATIONS**

Service Pack 3 has not been translated and is available in English only.

### **Changes for Service Pack 1**

- Updated text reports to include header and history section.
- Corrected issue with clinical link between BreezeSuite and Welch Allyn CardioPerfect ECG software.
- Corrected issue with blank pages on a GX report.
- Corrected issue with PF/GX trend data not updating in Breeze Review in between reviewing patient visits.
- Corrected issue with adjusting dead space values on GX scripts.
- Improved interface between FOT and BreezeSuite.

### **Changes for Service Pack 2**

- Integration of X-Scribe 4.0 CPI (cardiopulmonary interface).
- Corrected issues with active directory integration.
- Corrected issues with FOT graphs and labeling.
- Updated default FOT report and screen configuration graphs.
- Corrected issue related to auto filters and staff login settings.
- Corrected issue with Breeze Order Entry that allows users to cancel orders regardless of current order status.

### **Changes for Service Pack 3**

- Corrected issue with ExFVL default graph configuration.
- Corrected assert error issue indicating decrypted passwords do not exist in database if password entries are different between local and shared database.
- Corrected issue with External Device Calibration record when refreshing local database.
- Corrected issues related to DBTools identifying active directory server based on domain.