



# BREEZESUITE™ 7.2 Service Pack 7 Release & Installation Notice

This software is a Service Pack for BreezeSuite™ cardiopulmonary diagnostic software version 7.2 and is intended to be installed on any computer that has BreezeSuite 7.2 already installed. BreezeSuite 7.2 Service Pack 7 can be installed on workstations, but not on BreezeSuite Servers.

## Minimum System Requirements

- BreezeSuite 7.2

**Warning:** Installing Service Pack 7 over BreezeSuite 7.2 will over-write the translation files. Reapply the translations by following the directions below.

**NOTE:** For multiuser installations, SP7 is only applied to workstations. SP7 is not applied to the BreezeSuite Multiuser Server installation.

## INSTALLATION

1. Close all applications.
2. Insert CD into CD Drive. (If downloaded from the MGC Diagnostics website, extract/unzip the BreezeSuite72SP7.zip file and select BreezeSuite72SP7.exe.)
3. Wait for the installation to start automatically (~15 seconds) or run the BreezeSuite72SP7.exe file on the CD.
4. After the software unzips the files it will begin to install the extracted files to your hard drive.
5. After the files have been copied remove the CD.

## LANGUAGE TRANSLATIONS

Service Pack 7 has been translated into: French, German, Italian, Spanish, Portuguese, Polish and Russian

1. Follow the Installation Instructions above before proceeding
2. With the SP7 CD in the CD Drive, browse to the **Translations** folder on the CD.
3. If the files were downloaded from the MGC Diagnostics website, browse to the **Translations** folder of the file that was extracted.
4. Double click on the appropriate language executable file.
5. At the "Location To Save Files" screen select **Next**.
6. At the "Overwrite Protection" screen select **Yes To All**.
7. After the files have been extracted and copied select **Finish**.

## Changes from previous Service Packs

- Modified Breeze Review to streamline the review function. Patient visits can now be listed in the main screen. Multiple patients can be reviewed without going back to the main screen.
- Changed the writing of a debug file from the root directory.
- Added "30 Second Rolling" GX Averaging

- Improved importation of SensorMedics patient files
- Corrected issue for Bronchial Provocation and Trend graphs in Breeze Review
- Added shaded predicted area to Static Compliance graph
- Changed the lower limit of CO gain for RTD systems to 0.600
- Improved the deadspace detection in DLco test for the RTD systems
- Changed the lower limit of the CO<sub>2</sub> gain calibration factor to 0.500 for Ultima Tab 006 systems
- Corrected the error “Invalid CD Key” after software installation.
- Corrected calculation for CHO, Fat and Protein gms/day

#### **Change for Service Pack 7**

- Corrected issue where if selecting “Update Query Database”, an MFC Application error would be displayed and the program does not complete the update.