



BREEZESUITE™ 8.2 Service Pack 3 Release & Installation Notice

This software is a Service Pack for BreezeSuite™ cardiorespiratory diagnostic software version 8.2 and is intended to be installed on any computer that has BreezeSuite 8.2 already installed. BreezeSuite 8.2 Service Pack 3 can be installed on workstations, but not on BreezeSuite Servers.

Minimum System Requirements

- BreezeSuite cardiorespiratory diagnostic software version 8.2

Warning: Installing Service Pack 3 over BreezeSuite software version 8.2 will over-write the translation files.

NOTE: For multiuser installations, Service Pack 3 is only applied to workstations. Service Pack 3 is not applied to the BreezeSuite Multiuser™ Server installation.

INSTALLATION

1. Close all applications.
2. After downloading from the MGC Diagnostics website, extract/unzip the BreezeSuite82SP3.zip file.
3. Double click on the BreezeSuite82SP3.exe file from the extracted files.
4. After the software installation is run, it will begin to install the appropriate files to your hard drive.
5. After the software installation has been completed, select Finish.

LANGUAGE TRANSLATIONS

Service Pack 3 has been translated into: French, German, Italian, Spanish, Portuguese, Polish and Russian.

1. Follow the Installation Instructions above before proceeding.
2. Browse to the **Translations** folder from the extracted files.
3. Double click on the appropriate language executable file.
4. At the “Location To Save Files” screen select **Next**.
5. At the “Overwrite Protection” screen select **Yes To All**.
6. After the files have been extracted and copied select **Finish**.

Changes for Service Pack 1

- Addresses communication issues with Mortara X-Scribe software interface “ECG Only” testing in Ultima Cardio₂ configurations

Changes for Service Pack 2

- Corrected an intermittent issue where the treadmill would go to 0.5 mph instead of zero when users click on the “Stop Treadmill” button
- Improved the speed of SUM file creation when exporting the SUM file, thus eliminating Instrument Manager errors

Changes for Service Pack 3

- Corrected issue with scissor valve operation
- Corrected communication issue related to Lode ergometers utilizing legacy or obsolete versions of Lode ergometer firmware/communications